

ABIOMED® BVS®5000
BI-VENTRICULAR SUPPORT SYSTEM
OPERATOR'S MANUAL

CAUTION

Federal (U.S.A.) law restricts this device to sale, distribution and use by or on the order of a physician.

For use only by personnel trained in accordance with the ABIOMED Training program.
Read all warnings, precautions, and instructions carefully prior to use.

Manufactured by: ABIOMED, Inc.
 22 Cherry Hill Drive
 Danvers, MA 01923 USA
 (978) 777-5410

24-Hour Emergency Hotline: N. America 1-800-422-8666

European Representative:
 ABIOMED Europe GmbH
 Neuenhofer Weg 3
 52074 Aachen, Germany
 +49 (241) 8860-0 (voice)
 +49 (241) 8860-111 (fax)
 (EU 24 Hour, Emergency Hotline)
 +49 (0)1805 2246633
 europa@abiomed.com (email)

0005-9006 Rev. P
September 2007

ABIOMED, Inc. and our authorized distributors are responsible for the effect of safety, reliability, and performance of the equipment only if:

All service, repairs, and periodic maintenance are carried out by
ABIOMED, Inc. or its authorized representative or distributor.

The electrical environment and installation of the User's site complies
with all VDE requirements (if applicable).

The BVS 5000 is only used in accordance with the Instructions for Use.

Copyright 1987, 1989, 1992, 1993, 1997, 2000, 2001, 2002, 2006

ABIOMED, Inc.

All rights reserved.

“ ABIOMED, BVS and Angioflex are trademarks of ABIOMED, INC. and are registered in the U.S.A. and certain foreign countries. AB5000 is a trademark of ABIOMED, INC.

TABLE OF CONTENTS**SECTION 1. INTRODUCTION**

1.1	General	4
1.2	BVS Blood Pump	4
1.3	BVS Console	5
1.4	System Use	6
1.5	Accessories	6
1.6	BVS Console Specifications	6
1.7	BVS Blood Pump Specifications	9

SECTION 2. OPERATION

2.1	Console Power Up and Self Test	11
2.2	BVS Blood Pump Preparation	13
2.3	Recommended Cannulation Method	13
2.4	Interconnection Procedure	13
2.5	Pumping	14
2.6	Battery Operation and Transport	15
2.7	Weaning	19

SECTION 3. ALARMS AND STATUS INDICATORS

3.1	Introduction	21
3.2	Flow and Pressure Alarms	21
3.3	Alarm Mute	21
3.4	Battery Status	22
3.5	Emergency System	22

SECTION 4. COMPLETE CONSOLE FAILURE

4.1	Foot Pump	23
4.2	Removing and Operating the Foot Pump	23
4.3	Stowing Foot Pump	24

SECTION 5. ROUTINE MAINTENANCE AND SHIPMENT

5.1	Routine Check of Backup Systems	25
5.2	Foot Pump Test	25
5.3	Self-Test Routine	25
5.4	Emergency System Test	25
5.5	Packing for Shipment	25
5.6	Cleaning	26
5.7	Scheduled Maintenance	26
5.8	BVS Replacement Parts	26
5.9	Service	27
5.10	Ordering Information	28

BVS CONSOLE DISPLAY MESSAGES	29
-------------------------------------	----

OPERATING INSTRUCTION PLACARD	31
--------------------------------------	----

APPENDIX A. SPECIAL CONSIDERATIONS WHEN CHANGING TO AN AB5000™ CONSOLE	32
---	----

SECTION 1. INTRODUCTION

1.1 General

The ABIOMED BVS 5000 Bi-Ventricular Support System is an automated bi-ventricular support device intended to provide complete short term support of the left and/or right sides of the heart. The system uses a microprocessor-based drive console to supply pneumatic power to a disposable blood pump. Beat rates and systolic/diastolic intervals are determined automatically by sensing drive line air flow at the console. The console drives and adjusts the left and right blood pumps independently of each other.

1.2 BVS Blood Pump

The disposable BVS Blood Pump (Figure 1-1) is a pneumatically driven two-chambered device which supports one side of the heart. The pump is placed external to the patient. Blood inflow from the left atrium is returned to the thoracic aorta. Inflow from the right atrium returns to the pulmonary artery. Connection of the blood pump to the patient uses Tran thoracic Cannulation techniques.

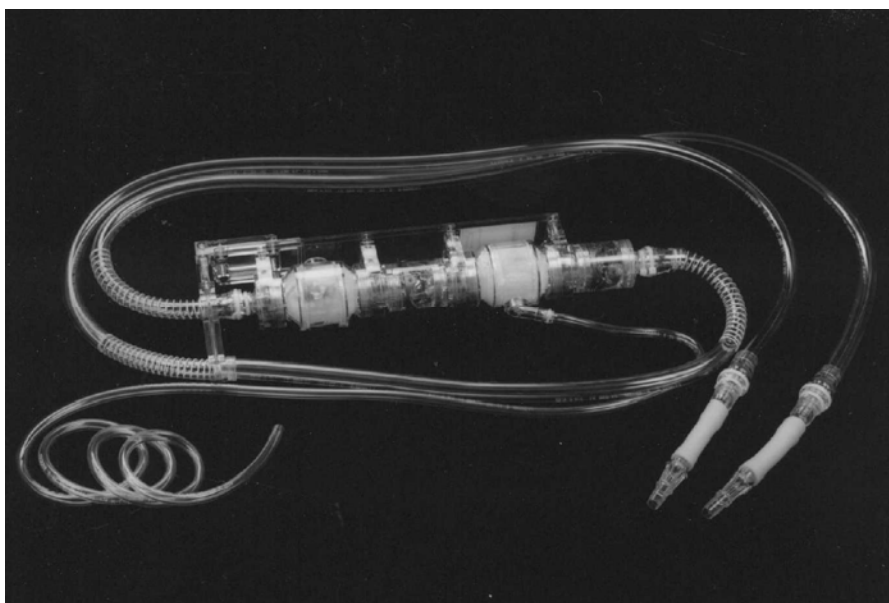


Figure 1-1. BVS Blood Pump

There are two cannulae supplied with each blood pump; one for left and right atrial connections, and one for the aortic or PA connection. The atrial and arterial cannulae are both supplied with tie wraps, a tunneling bullet for tunneling the cannula through the skin, and a cannula restraint for the cannula connector junction. The arterial cannula is manufactured with a coated 10mm or 12mm precoated graft bonded to one end. This graft permits end-to-side anastomosis of the cannula to the PA or aorta. The elastomeric coating on the graft **eliminates the need for preclotting**, and provides a graft porosity $<1\text{cc}/\text{min}/\text{cm}^2$.

1.3 BVS Console

The BVS Console (Figure 1-2) is a microprocessor-controlled pneumatic drive system which can operate one or two disposable blood pumps. The console automatically adjusts the beat rate and systolic/diastolic ratio based on air flow into and out of the blood pump. Blood pump flow and rate are displayed on the control panel.



Figure 1-2. BVS System 5000 Console

The console incorporates several systems to ensure safe operation. Automatic self-testing upon power up alerts the operator about potential problems while operator observes the control panel during the test procedure. Battery backup is provided for one hour (when fully charged) of portable or power fail operation. An alphanumeric display and message indicators show BVS system status. Alarms alert the operator to abnormal flow or pressure conditions.

The console has two levels of operational redundancy. Should the microprocessor-based system fail, a hardware-based backup system continues pumping at a fixed rate. In case of complete console failure, a foot-operated pump allows continued support.

Special considerations must be taken when swapping between a BVS[®] 5000*i* console and an AB5000[™] console for patients supported on a BVS[®] Blood Pump. Under certain conditions, these consoles may display different flow rates. Refer to Appendix A for a description of these considerations.

1.4 System Use

The first step in the use of the BVS 5000 is powering up the console. This should be done well before blood pump priming to permit performance of the automatic self-test well in advance of use. The sterile blood pump is primed and connected to the console. This is typically done while awaiting completion of surgical cannulation. The blood pump tubing and cannulae are purged of air and connected. The blood pump is mounted at the appropriate level relative to the patient. After the console starts pumping, it adjusts its beat rate and systolic/diastolic ratio as required to optimize pumping.

When the patient needs to be transported, the blood pump can be mounted to the bed with the BVS Bed Mount or can remain in the BVS I.V. Pole Mount. The console is simply unplugged and its line cord stowed. The internal battery provides one hour of operation when fully charged.

To wean the patient from the BVS system, weaning controls can be set for the desired flow. The system will automatically limit its output to the selected flow.

1.5 Accessories

The BVS I.V. Pole Mount is provided as a mounting bracket for the BVS Blood Pump while in use. It attaches to an I.V. pole and can accommodate two blood pumps.

The BVS Bed Mount is also an accessory that can be used to secure the blood pump during patient transport. It slides between the mattress and can accommodate two pumps. This obviates the need for transporting the pumps on an I.V. pole next to the patient.

1.6 BVS Console Specifications

1.6.1 Physical

Width:	24 inches (61 cm)
Depth:	22 inches (56 cm)
Height:	33 ¼ inches (84.5 cm)
Weight:	180 lbs. (81.6 kg)
Operating Temperature Range:	10°C to 40°C
Operating Humidity Range:	30% to 75% (non-condensing)

1.6.2 Electrical

Mains Voltage:	Factory configured for operation from either 100 VAC, 120 VAC, 200 VAC, 220 VAC, or 240 VAC.
Mains Frequency:	47 to 63 Hz
Mains Current:	3.6 A maximum at 120 V, 1.8 A maximum at 220 V.
Average power consumption:	280 W.
Line Leakage Current:	Less than 100 uA at any input voltage.
Power Cable:	11 ft. (3.4m) of #16 AWG Type SJT; hospital grade plug (N.A.). 3.1 m harmonized 3 x 1.00 mm ² , 10 A/250 V (European).
Battery:	Internal 24 V battery provides 1 hour operating time when fully charged. Charging time is 16 hours to full charge after 1 hour of use.

1.6.3 Mechanical System

Drive medium:	Compressed air.
Drive Technology:	Compressor system drives independent pressure-regulated systems. Switching from a pressurized to a non-pressurized state in the blood pump is via independent 3-way solenoid valves. Return air is allowed to vent to atmosphere; no vacuum is used. Left line pressure regulated to 250 mmHg. Right line pressure regulated to 200 mmHg.

1.6.4 Operator Controls

Pump Controls:	Pressing Left or Right "ON" button initiates pumping. Pressing Left or Right "OFF" button twice within 13 seconds (typical) ceases pumping.
Weaning Controls:	While pumping, the "SELECT" potentiometer is turned while depressing the "ACTIVATE" button. The desired flow is selected and the "ACTIVATE" button is released to engage weaning at the selected flow rate.
Alarm Mute Switch:	Silences audible alarm for 1 minute (except for complete console failure, which has separate alarm).
Operator Display:	4 line by 40 character LCD with electroluminescent backlighting.

1.6.5 Outputs and Inputs

LEFT and RIGHT device output: Blood pump interface 1/4 in (6mm) I.D. hose barb.

SERVICE ONLY – MONITOR input: DB15 connector for ABIOMED keyboard monitor assembly.

 **Used for troubleshooting by ABIOMED authorized service personnel only.**

1.6.6 Pumping System Control

General: Filling and emptying of disposable is controlled by computer examination of air flow and volume into and out of the blood pump.

Flow Rate: Calculated by the computer system integrating the area under the air flow curve and performing a four-beat weighted average of the results.

Pumping Rate: With pump height as specified in Section 1.7:
 Left side: >4.0 L/min at outflow pressures of 90 mmHg and inflow pressure of >10 mmHg.
 Right side: >4.0 L/min at outflow pressure of 40 mmHg and inflow pressure of >10 mmHg.

Weaning Method: Rate reduction via computer control.

Weaning Range: 0.5 L/min to full available flow.

1.6.7 Alarm Limits

LOW FLOW: <1.8 L/min flow. (<0.5 L/min flow in weaning)

LOW PRESSURE: Left side: <100 mmHg during systole.
 Right side: <50 mmHg during systole.

HIGH PRESSURE: Left side >240 mmHg during systole.
 Right and Left side >50 mmHg during diastole.
 Right side >190 mmHg during systole.

BATTERY ON LOW: Less than 10 minutes of battery time remaining. (This is the minimum time remaining when "LOW" first comes on.)

BATTERY CHARGING: Displayed when battery is below 80% capacity.

1.6.8 Hardware Backup System

Indication: EMERGENCY SYSTEM ON indicator.

Allowable Low Limit: Backup engages if system pumping <3 BPM.

Allowable High Limit: Backup engages if system pumping >140 BPM (typical).

Backup Pumping Rate: 33 - 39 BPM, 50% systole, open loop control.

1.7 BVS Blood Pump Specifications

<u>Parameter</u>	<u>Specifications</u>
No. of bladders	Two: filling and pumping
No./type of valves	Two: inflow and outflow, ABIOMED, trileaflet Angioflex® Polyurethane
Bladder/valve material	Angioflex Polyurethane
Volume of pumping bladder	100 cc
Volume of filling bladder	100 cc
Extracorporeal volume (system)	<660 cc
Operating pressures on bladder	Left side: 250 mmHg Right side: 200 mmHg
Bladder burst pressure	>325 mmHg
Valve prolapse pressure	>350 mmHg
Valve forward flow ΔP	<6.5 mmHg at 10 L/min
Drive medium applied to bladder	Compressed air
Housing material	Class VI medical grade polycarbonate
<i>Patient connections:</i>	
Blood pump inflow	48 inches of 1/2 inch I.D., 3/32 inch wall thickness Class VI surgical grade PVC tubing with 1/2 inch to 1/2 inch Class VI polycarbonate tubing connector attached.
Blood pump outflow	72 inches of 1/2 inch I.D., 3/32 inch wall thickness Class VI surgical grade PVC tubing with 1/2 inch to 1/2 inch tubing connector attached.

Atrial Cannula	32 French reinforced malleable backbone supplied with cannula restraint, tunneling bullet, and tie wraps.
	42 French reinforced malleable backbone supplied with cannula restraint, tunneling bullet, and tie wraps.
Arterial Cannula	12mm reinforced, supplied with cannula restraint and tunneling bullet, with integral precoated graft attached.
	10mm non wire reinforced, supplied with cannula restraint and tunneling bullet, with precoated <i>Hemashield</i> ™ graft attached.
Drive console connections	72 inches (2m) of PVC Class VI tubing 1/4 inch I.D., attached to the blood pump for console interface. It should be connected to appropriate (left, right) hose barb on the console.
Sterilization	The disposables are ethylene-oxide gas sterilized, the blood pump is packaged in a <i>Tyvek</i> ™/PETG tray, and the cannulae in a <i>Tyvek</i> / <i>Mylar</i> ™ pouch. The 10mm <i>Hemashield</i> graft Arterial Cannula is gamma-sterilized.
Weight	3.5 lbs (1.6 kg)
Mounting methods	BVS blood pump can be mounted to an I.V. pole or vertical pole mount or to the bed with the appropriate selection of accessories.
Blood pump mounting height	42 Fr. 0-10 inches below the level of the patient's atrium.
	32 Fr. 4-14 inches below the level of the patient's atrium.
Storage conditions	10°C to 40°C, relative humidity 30 to 75%

SECTION 2. OPERATION

2.1 Console Power Up and Self-Test

If possible, the BVS Console should be powered up one hour before use; however, there is no minimum warm-up time required. A self-test procedure automatically runs after power up, and requires about one minute to complete. During self-test, the console flashes all indicators, sounds the alarm tone, and pumps air through the drive ports. It also checks the processor system integrity. **The operator should observe the control panel during the self-test to detect any abnormalities. Do not depress pump "ON" controls during self-test. This will result in activation of emergency system. (See Section 3.4).**

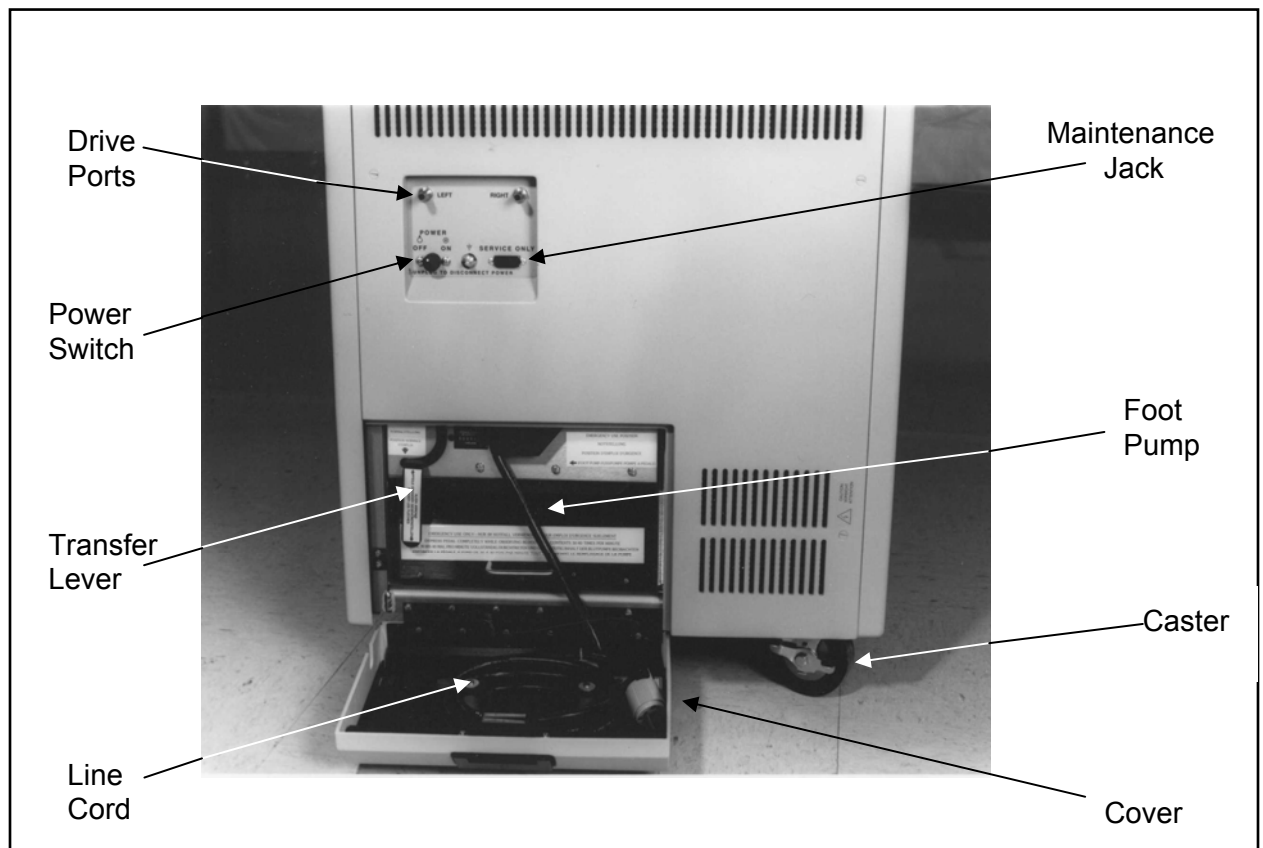


Figure 2-1. BVS Console Rear Panel

- Open the rear cover, (Figure 2-1), remove the line cord and plug into an outlet. (Check that CHARGING indicator is lighted on the front panel.) Close the cover while routing the cord out through the side opening provided. Open the weaning door. Activate the console with the power switch on the rear panel. The system will automatically begin to run its self-test.

⚠ Unplug line cord to disconnect all AC power from console.

b) Check visible and audible machine functions:

Watch for the flashing of all alarm and status lamps (Figure 2-2). Only the EMERGENCY SYSTEM ON and BATTERY indicators will not light. (Some BATTERY indicators may be lighted, depending on battery condition.)

Listen for the alarm tone. It first sounds loud and fast, then sounds softer and slower.

Listen for valve actuation and the sound of air venting from the rear panel drive ports.

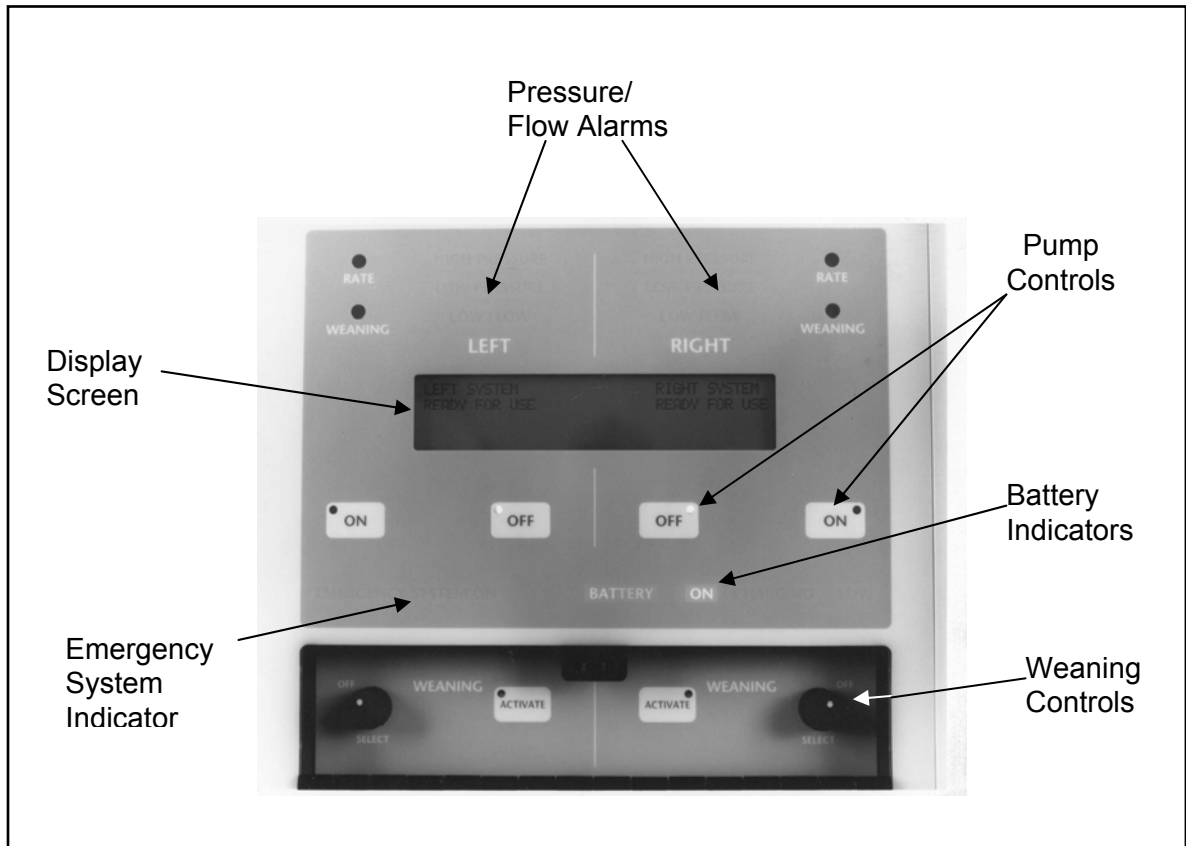


Figure 2-2. Control Panel with Indicators Lit

c) The display should read LEFT (RIGHT) SYSTEM READY FOR USE after self-test (Figure 2-3)



Figure 2-3. Display, System Ready for Use

If the display reads SELF TEST FAILED, (Figure 2-4) or if any message other than the "ready for use" message appears, call qualified service personnel immediately.

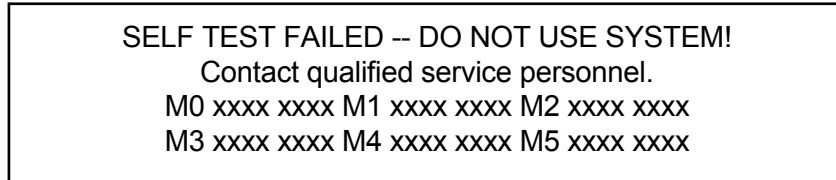


Figure 2-4. Display Self-Test Failure

NOTE: M0 xxxx xxxx may not be displayed, and M5 xxxx xxxx may be replaced with RAM xxxx xx xx.

- d) Check the condition of the batteries. If the BATTERY CHARGING indicators are illuminated, the batteries are not capable of providing a full hour's operation of the console. Allow more charging time before portable use.

The console is now ready for operation.

Caution: Do not lean on or place any objects on the control panel.

2.2 BVS Blood Pump Preparation

See BVS 5000 Bi-Ventricular Support System Blood Pump Set [Instructions for Use](#) prior to operation.

2.3 Recommended Cannulation Method

See BVS 5000 Bi-Ventricular Support System Blood Pump Set [Instructions for Use](#) prior to operation.

2.4 Interconnection Procedure


See BVS 5000 Bi-Ventricular Support System Blood Pump Set [Instructions for Use](#) prior to operation.

2.5 Pumping

After clamps placed on blood lines have been removed, press the ON button for each side used to begin pumping.

2.5.1 Expected Beat and Flow Rates

Observe the beat rate (Figure 2-5). Any gross instability of beat rate (double beats, etc.) indicates a possible problem with the placement of the blood pump relative to the patient (refer to Section 2.5.2).

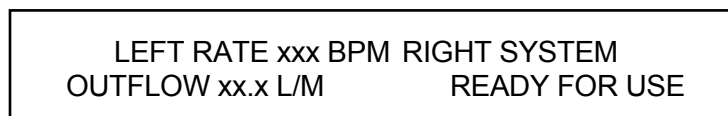

```
LEFT RATE xxx BPM RIGHT SYSTEM
OUTFLOW xx.x L/M      READY FOR USE
```

Figure 2-5. Display, Normal Pumping

A step change in rate is normal, especially during the first two minutes. During this period, the console adjusts itself to determine the optimal pumping rate. A brief extension of one beat every two minutes is normal, as the console periodically recalibrates itself.

2.5.2 Adjustment of Blood Pump Filling

Proper filling of the blood pump can easily be assessed through an examination of blood pump flow and the behavior of the upper bladder. When operating in the normal (full flow) mode, flow should be greater than 2 L/min, and typically ranges from 3 L/min to 5 L/min.

To optimize blood flow, first make certain that the patient is appropriately hydrated, with filling pressures within their normal ranges. If the patient is hypovolemic, administer fluids according to standard procedures. Second, observe the filling and emptying of the upper bladder. The bladder should just barely fill the plastic chamber during pump systole, and just empty during pump diastole. If the bladder is too full, move the blood pump higher to reduce inflow pressure. If the bladder is too empty and the patient is sufficiently hydrated, lower the pump to increase inflow pressure.

The recommended level of the upper bladder should be positioned between 0 and 14 inches (35 cm) below the patient's atria. If the pump is raised above the level of the patient's atria or is lowered more than 10 inches below the patient's atria, a decrease in pump flow may result. After adjusting blood pump height, observe bladder filling for two minutes before adjusting again. This will allow the system to reach steady state before further adjustment.

2.5.3 To Stop Pumping

To stop pumping, press the appropriate OFF button **twice within thirteen seconds** (typical). The button **must** be pressed twice to stop the pump (Figure 2-6). This is a safety feature to prevent accidental operation. A **single** press of the OFF button will be ignored after thirteen seconds (typical).

TO STOP PUMPING, PRESS "OFF" AGAIN	RIGHT SYSTEM READY FOR USE
---------------------------------------	-------------------------------

Figure 2-6. Display After Pressing OFF Button Once

2.6 Battery Operation and Transport

When the patient is transported from the operating room to intensive care, the blood pump can be mounted to the bed. The console is unplugged and rolled along with the patient. The internal battery, when fully charged, will automatically power the console for one hour.

The battery requires approximately 16 hours to charge from a totally drained state.

2.6.1 Mounting the Blood Pump to the Accessory Bracket

Two types of accessory brackets are available to mount the blood pump for transport (Figure 2-7). Each bracket can hold one or two blood pumps.

The BVS pole mount attaches to either a standard I.V. pole or the BVS bed mount that slips under the patient's mattress.

2.6.1.1 To attach the BVS I.V. Pole Mount to the I.V. pole:

- 1) Unscrew the screw clamp enough to fit the clamp around the I.V. pole.
- 2) Tighten the screw clamp onto the I.V. pole by turning the black knob clockwise.
- 3) Adjust the height of the Pole Mount by turning the central adjustment lever counter clockwise one half turn to loosen it, while holding the handle (instead of pointing down, the adjustment lever will now be pointing up toward 12 o'clock).
- 4) Move the Pole Mount up or down with the handle to the desired position.
- 5) Turn the central adjustment lever clockwise, back to its original position, to secure the Pole Mount in place.

2.6.1.2 To attach the BVS Pole Mount to the Bed Mount:

- 1) Hold the bed plate so the metal post faces up.
- 2) Slide the bed plate under the head of the mattress, (between mattress and bed) so that the metal post is facing up next to the mattress.
- 3) The weight of the mattress and the patient will hold the Bed Mount Plate in place.
- 4) Hold the Pole Mount by the handle.
- 5) Unscrew the screw clamp enough to fit the clamp around the metal post.
- 6) Tighten the screw clamp onto the metal post by turning the black knob clockwise.
- 7) Adjust the height of the Pole Mount by turning the central adjustment lever counter clockwise, one half turn to loosen it, while holding the handle.
- 8) Instead of pointing down, the central adjustment lever will now be pointing up toward 12 o'clock.
- 9) Move the Pole Mount up or down with the handle to the desired position.
- 10) Turn the central adjustment lever clockwise, back to its original position, to secure the Pole Mount in place.

2.6.1.3 To attach the BVS Blood Pumps to the Pole Mount:

- 1) A long plastic plate is attached to the back of the blood pump.
- 2) At the top of this plate, slightly above the upper inflow bladder, is a square backplate designed to slide into the blood pump bracket.
- 3) Hold the blood pump securely so the inflow and outflow tubing is at the top of the blood pump and the drive line is at the bottom of the blood pump.
- 4) Slide the backplate into one of the blood pump brackets from the top down.
- 5) Make sure that both edges of the blood pump bracket fully engage the blood pump backplate to ensure that it is attached securely.

2.6.1.4 Attaching the BVS Blood Pumps to the console:

- 1) Connect the air hose from the blood pump to the respective right or left hose connector on the back of the console (Figure 2).

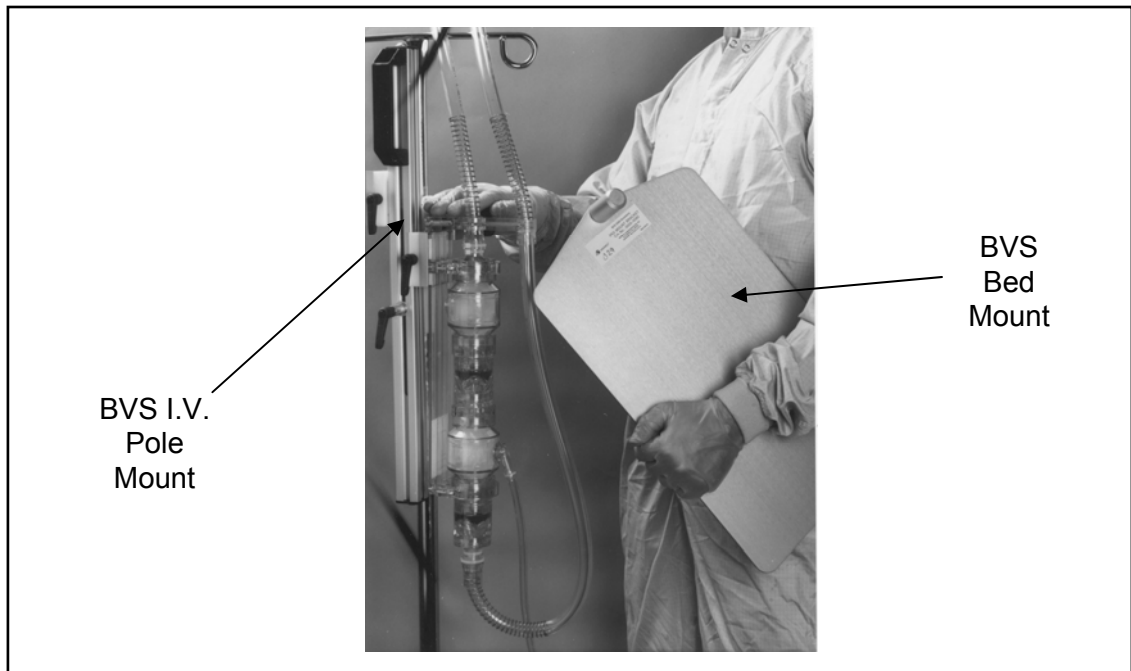


Figure 2-7. Blood Pump accessory mounting brackets

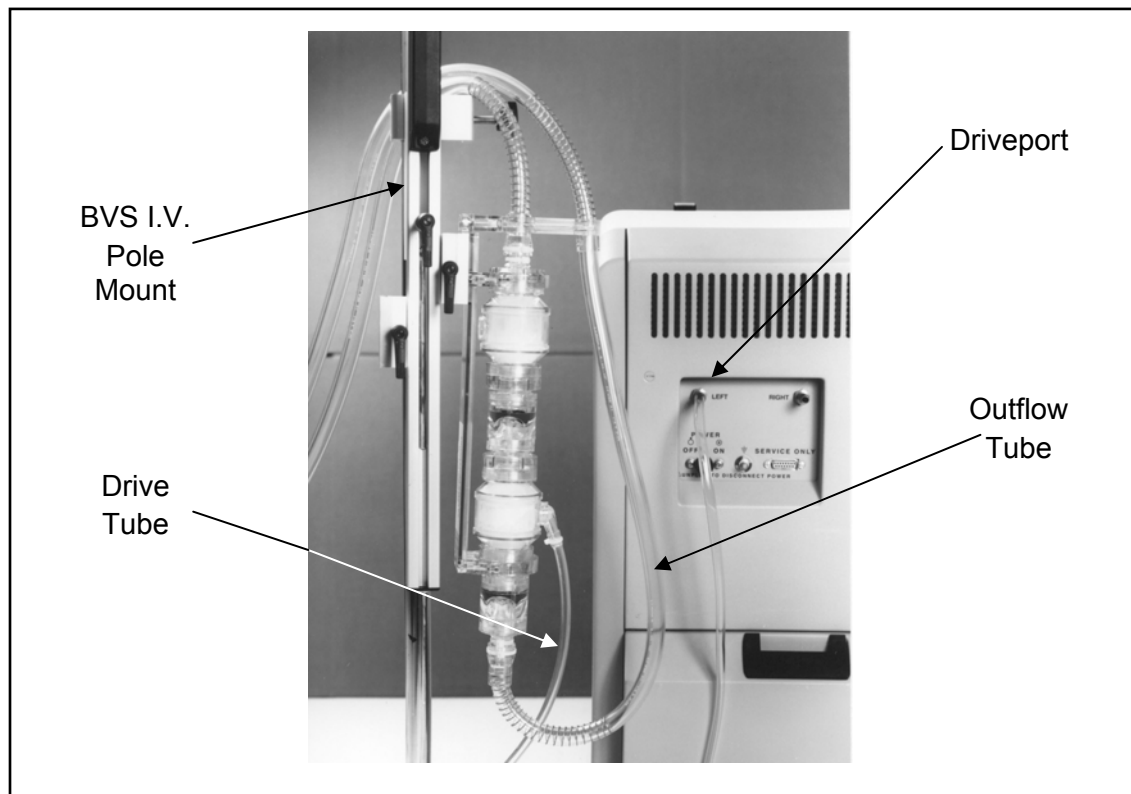


Figure 2-8. Blood Pump Hooked up to Console

2.6.2 Battery Status

As long as the console line cord is plugged into an AC power source, the internal battery will be kept in a charged state. A row of indicators below the control panel (Figure 2-9) shows the status of the battery.

No indicators are illuminated when the console is plugged in and turned on, and the battery is over 80% charged.

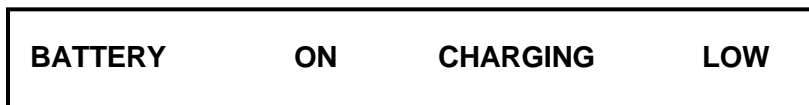


Figure 2-9. Battery Status Indicators

BATTERY CHARGING appears when the console is plugged in and turned on, and the battery charge is less than 80% capacity.

BATTERY ON appears when the line cord is disconnected and the console is operating on its battery. A periodic two-beep audible signal also indicates that the battery is on.

BATTERY ON LOW appears when the line cord is disconnected and less than 10 minutes (minimum) of battery operation remains. A loud periodic four-beep audible signal is produced to advise the operator of low battery status. The operator should either plug in the console line cord or obtain a backup console.

CHARGING (without the word BATTERY) appears when the console is plugged in, but turned off. This indicator is not related to the charge state of the battery; it simply indicates the availability of power to charge the battery if needed, or to maintain a fully charged condition.

2.6.3 Preparing Console for Transport

Battery operation is automatically activated by unplugging the line cord. When ready for transport, open the rear cover, unplug the line cord, and wind it around the cord carrier. Close the cover. Roll the console carefully to avoid kinking or disconnection of the drive line.

To carry the console over obstacles, grasp the recessed handholds **under** the sides of the top panel. Make certain to lift the console by the top of the side panels and not by the top panel. Be careful that your hand does not get pinched between the top and side panels.

Caution: Do not attempt to lift the console by the top panel. Equipment damage may result.

2.7 Weaning

When the patient is to be weaned from the BVS system, the blood pump output may be set at any desired flow from 0.5 L/min to full flow (in 0.1 L/min increments). The operator simply selects the desired flow rate and the console reduces its pumping rate to achieve the desired flow. Whenever the weaning controls are activated, a periodic audible beep will be issued as a reminder. If flows drop below 0.5 L/min, a continuous audible alarm will be activated.

2.7.1 Weaning Adjustment

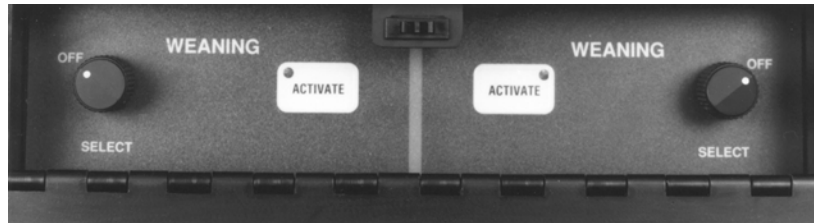


Figure 2-10. Weaning Control Panel

To begin weaning, open the weaning door. Turn the appropriate SELECT knob clockwise (Figure 2-10). The WEANING indicator will light. The ACTIVATE light will begin to blink, and the ACTIVATE message will appear on the display (Figure 2-11).

To set flow, press and hold ACTIVATE	RIGHT SYSTEM READY FOR USE
---	-------------------------------

Figure 2-11. Display, Weaning Activation

Press the ACTIVATE button and turn the SELECT knob to set the desired blood flow (Figure 2-12). The setting of the SELECT knob will appear on the display as long as the ACTIVATE button is being pressed.

Turn SELECT to set FLOW: xx.x L/M	RIGHT SYSTEM READY FOR USE
--------------------------------------	-------------------------------

Figure 2-12. Display, Weaning Adjustment

When the ACTIVATE button is released, the console will begin to adjust itself toward the new weaning flow (Figure 2-13). This will generally take at least 8 beats.

LEFT RATE xxx BPM WEANING @ xx.x L/M	RIGHT SYSTEM READY FOR USE
---	-------------------------------

Figure 2-13. Display, Weaning Operation

To return to normal operation, turn the SELECT knob counter-clockwise to the OFF position. The console will automatically return to full available flow.

This page left intentionally blank

SECTION 3. ALARMS AND STATUS INDICATORS

3.1 Introduction

The BVS console provides alarms for high or low pumping pressure and low blood flow. These alarm messages appear on the display screen, and on the panel above the display. Status indicators of the emergency pumping system and battery system appear below the pump controls.

3.2 Flow and Pressure Alarms

The low flow and high or low pressure alarms (Table 3-1) are derived from the pressure of the air line to the blood pump. The air line pressure and flow are measured inside the BVS Console. Blood flow is calculated based upon these measurements. Flow and pressure alarms will typically indicate a blocked air or blood line. They may also indicate blood pump placement at the wrong height relative to the patient.

Alarm Message	Probable Causes
LOW FLOW	<ul style="list-style-type: none"> • obstruction of blood lines • blood pump placed too high relative to atrium • inadequate blood volume
HIGH PRESSURE LOW FLOW	<ul style="list-style-type: none"> • drive line or blood pump lines kinked or occluded
LOW PRESSURE LOW FLOW	<ul style="list-style-type: none"> • disconnection or leak in drive line

Table 3-1. Flow and Pressure Alarm

Caution: Obtain a backup console if all of the above have been checked and these alarms continue to occur with pump rates at 20 to 25 BPM in the "normal" operation mode.

3.3 Alarm Mute

Pressing the alarm mute switch silences the audible alarm for one minute but does not affect the visual display of alarm conditions. A yellow light indicates that the alarm mute is activated.

At the end of the minute, the alarm will be enabled again, without operator intervention. The mute interval will be resettable, but not cumulative. This means that the operator can press the mute button one or more times during the one minute mute period, and the mute time will always be reset to one minute from the last activation of the button; it will not add one minute to the time remaining from the previous mute period. Continuous depression of the alarm mute switch does not extend the mute duration.

3.4 Battery Status

As long as the console line cord is plugged into an AC power source, the internal battery will be kept in a charged state. A row of indicators below the control panel shows the status of the battery (Table 3-2). No indicators are illuminated when the console is plugged in and turned on, and the battery is over 80% charged.

The console will periodically issue a two-beep audible indication while operating on battery. If the battery system enters the LOW condition, the indication becomes a louder four beeps.

Message	Cause	Action
BATTERY CHARGING	Battery less than 80% charged	Leave console plugged in until battery is charged
BATTERY ON	Line cord disconnected and console running on battery.	Expect less than one hour operation on battery power
BATTERY ON LOW	Less than 10 minutes of battery operation remain.	Plug in console or obtain a backup
CHARGING	Normal condition when console is turned off, and plugged into AC power source	None required

Table 3-2. Battery Status Message

3.5 Emergency System

If the microprocessor-based pump control system fails, a fixed rate control system takes over. This system will continue pumping at approximately 35 BPM and will light the Emergency System indicator (Table 3-3).

When the emergency system is on, only the rate and battery indicators are valid. The ON and OFF buttons function normally, but their indicator lights should be ignored. If the emergency system was engaged by the direction of the processor, a continuously pulsing audible alarm will sound.

Note: Depression of pump on switches during self-test is intended to activate emergency system. This is normal operation. To reset system, turn power off then on.

Message	Cause	Action
EMERGENCY SYSTEM	Microprocessor system Failure	Obtain backup system as soon as possible. Call service personnel.

Table 3-3. Emergency System Message

Section 4. Complete Console Failure

4.1 Foot Pump

In the case of complete console failure, the foot-operated pump allows manual operation of the blood pump. The foot pump is located behind the console line cord storage compartment door. It can be removed for use after operating the transfer valve lever in front of it. A complete console failure is indicated by no air being pumped regardless of how the controls are operated.

4.2 Removing and Operating the Foot Pump

- a) Turn off the console. Open the rear cover.
- b) Move the transfer lever up 1/4 turn counter-clockwise to the horizontal position. Grasp the foot pump by hand, squeeze the pedal closed and pull it out of the slot, as shown in Figure 4-1.

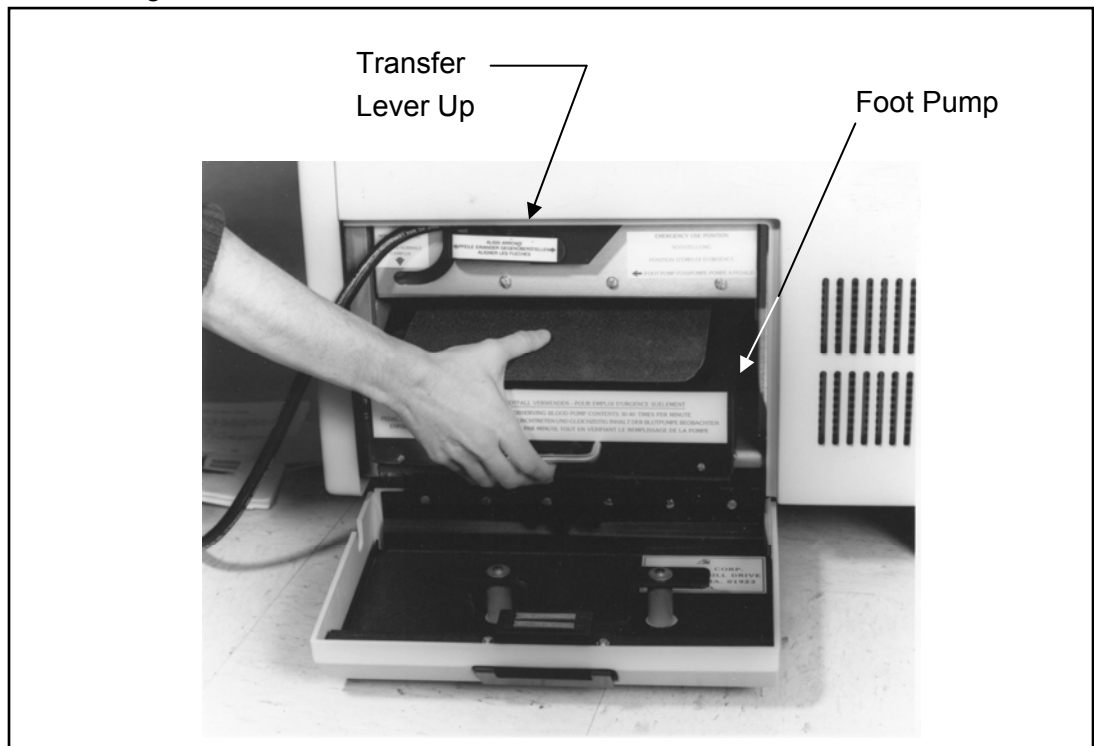


Figure 4-1. Removing Foot Pump

- c) Place the pump on a level floor. Make sure the hoses are not kinked.
- d) Completely depress the pump with your foot (Figure 4-2). **Allow it to fully return to its original position.** Repeat. This should result in a normal pumping rate of 30 to 40 times per minute. Observe the blood pump for proper filling and emptying. If the console is restarted, use of the foot pump must be stopped during the first four seconds of the self test.

- e) Obtain a backup BVS Console. Power it up and allow the self-test to run. Begin pumping by pressing the ON button. Transfer the drive line from the failed console to the backup.
- f) Call service personnel at the earliest opportunity to repair the failed console.

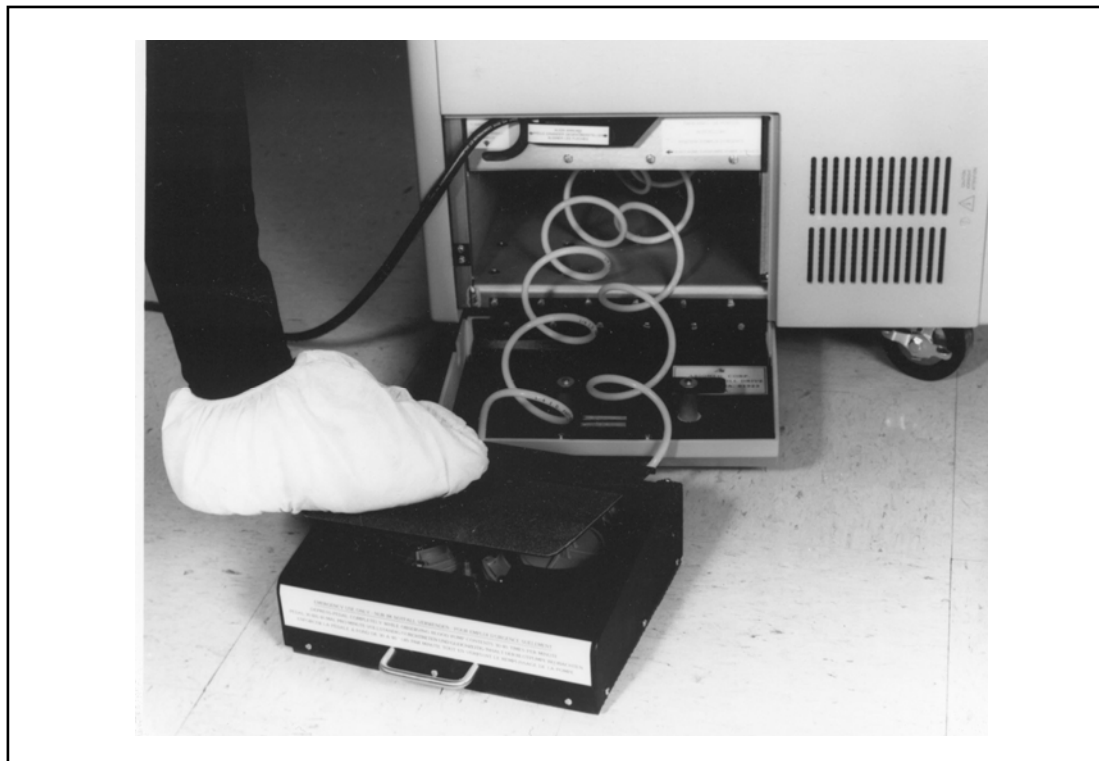


Figure 4-2. Operating Foot Pump

4.3 Stowing Foot Pump

- a) Guide the coiled hoses into the slot. Squeeze the pump closed and slide it into the slot behind the hoses.
- b) Swing the transfer lever down 1/4 turn clockwise to engage the console pump.

Note: The transfer lever must be turned back down or the console will not operate. If the lever is not turned back down fully, a message of FOOT PUMP HANDLE DISLODGED will appear on the display.

SECTION 5. ROUTINE MAINTENANCE AND SHIPMENT

5.1 Routine Check of Backup Systems

It is recommended that the BVS Console emergency systems be checked out regularly. This should be done on the day previous to scheduled surgery, and during preventive maintenance. The procedure is to test the foot pump, run the regular power up self-test, and activate the emergency control system.

5.2 Foot Pump Test

With the unit off, open the rear cover. Remove and set up the foot pump (refer to Section 4). Operate the pump and check that air flows out of both nozzles. Replace the foot pump. Make certain that the transfer lever is returned to the down position after testing is complete. If it is not, FOOT PUMP HANDLE DISLODGED will appear on the display.

5.3 Self-Test Routine

Remove the line cord and plug into an outlet. Open the weaning door. Turn on the power switch. Check the flashing panel indicators. Assure that all indicator lights work and that the alarm tone sounds. Listen for valve actuation, and for the sound of air venting from the rear panel pump connections.

The display should read LEFT (RIGHT) SYSTEM READY FOR USE. If any display other than this appears, such as SELF TEST FAILED, consider the unit inoperable. Obtain a backup unit and call qualified service personnel immediately.

Refer to Section 2.1 for more details.

5.4 Emergency System Test

Shut off the power switch in back of the unit. Then turn on power. During the self test period, press one of the ON buttons for approximately 2 seconds. Wait for approximately 90 seconds from the end of self test for the emergency system to activate. The EMERGENCY SYSTEM ON message and the RATE lamp should blink as the console pumps. Turn off the console power switch to deactivate the emergency system.

5.5 Packing for Shipment

If it becomes necessary to ship the BVS Console a long distance by vehicle, it should be shipped only in its original shipping crate. If the original crate is not available, contact the ABIOMED facility for a substitute crate and packing instructions.

5.6 Cleaning

The BVS Blood Pump can only be cleaned by the use of soap and water on all accessible parts. The console display/control panel area should be cleaned with isopropyl alcohol and cotton. It is recommended that the display window only be cleaned this way or with soap and water to avoid scratching. Use of a mild detergent with a wet sponge is suggested for the console enclosure if necessary.

Note: Do not allow the following agents to come in contact with the BVS Blood Pump because they may attack the plastic and cause damage to the device: halogenated hydrocarbon-based anesthetic agents, other halogenated hydrocarbons such as chloroform, highly alkaline chemicals such as sodium hydroxide, aromatic hydrocarbons such as gasoline, and ketones such as acetone.

5.7 Scheduled Maintenance

The BVS console should be installed and maintained by ABIOMED, Inc., an authorized service agent, or distributor only. The periodic maintenance schedule calls for:

<u>Maintenance</u>	<u>Period</u>
Perform cleaning, functional test, and calibration	1 year
Replace batteries.	2 years
Replace solenoid valves.	2500 hours of use
Replace compressor.	7000 hours of use
Replace pressure regulators and filters.	5000 hours of use
Estimated useful console life	8500 hours of use

Please contact the authorized distributor if the hospital record does not show that this schedule has been maintained.

5.8 BVS Replacement Parts

Contact your authorized ABIOMED representative for spare parts list.

Service

For service, contact:

ABIOMED, Inc.
22 Cherry Hill Drive
Danvers, Massachusetts 01923

(978) 777-5410
(800) 422-8666

5.10 Ordering Information**BVS Equipment:*****Standard Flow***

Catalog No.	Language	Mains voltage
0005-0000-E1	English	120VAC
0005-0000-E2	English	220 or 240VAC (specify)
0005-0000-E3	English	100VAC
0005-0000-F1	French	120VAC
0005-0000-F2	French	220 or 240VAC (specify)
0005-0000-G1	German	120VAC
0005-0000-G2	German	220 or 240VAC (specify)

Increased Flow

Catalog No.	Language	Mains voltage
0005-0010-E1	English	120VAC
0005-0010-E2	English	220 or 240VAC (specify)
0005-0010-E3	English	100VAC
0005-0010-F1	French	120VAC
0005-0010-F2	French	220 or 240VAC (specify)
0005-0010-G1	German	120VAC
0005-0010-G2	German	220 or 240VAC (specify)

Transport/Backup

Catalog No.	Language	Mains voltage
0050-0000	English	120VAC

<u>Catalog Number</u>	<u>Description</u>
0005-0090	BVS I.V. Pole Mount
0005-0080	BVS Bed Plate Mount
0005-0060	BVS Pump Mounting Set
0505-3170	Tubing Covers
0050-3200	Blood Pump Sling

BVS Disposables

<u>Catalog Number</u>	<u>Description</u>
Blood Pump Sets	
	Each blood pump set includes 1 blood pump, 1 atrial cannula, 1 arterial cannula and 3 tubing covers.
0505-0032	BVS Blood Pump Set - 32 Fr. (10.7mm) Atrial / 12mm graft Arterial Cannula
0505-0042	BVS Blood Pump Set - 42 Fr. (14mm) Atrial / 12mm graft Arterial Cannula
0505-0032-H	BVS Blood Pump Set 32 Fr. (10.7mm) Atrial / 10mm graft Arterial Cannula
0505-0042-H	BVS Blood Pump Set - 42 Fr. (14mm) Atrial / 10mm graft Arterial Cannula
0505-0032-MH	BVS Blood Pump Set - 32 Fr.M-VA (10.7mm) Atrial / 10mm graft Arterial Cannula
0505-0042-MH	BVS Blood Pump Set - 42 Fr.M-VA (14mm) Atrial / 10mm graft Arterial Cannula

<u>Catalog Number</u>	<u>Description</u>
Cannulae	
0506-0032-1AT	32 Fr. (10.7mm) Atrial (Inflow) Cannula
0506-0032M-VA	32 Fr.M-VA (10.7mm) Atrial (Inflow) Cannula
0506-0042-AT	42 Fr. (14mm) Atrial (Inflow) Cannula
0506-0042M-VA	42 Fr.M-VA (14mm) Atrial (Inflow) Cannula
0506-0042-AR	12mm graft Arterial (Outflow) Cannula
0506-0010-HAR	10mm <i>Hemashield</i> graft Arterial (Outflow) Cannula

BVS CONSOLE DISPLAY MESSAGES

<u>Self-Test Messages</u>	<u>Action</u>
SELF TEST FAILED -- do not use system! Contact qualified service personnel	Get backup console. (Section 2.1)
Visual/Audio check of machine functions: CHECK AUDIBLE SIGNALS or	Look and listen. (Section 2.1)
CHECK ALARM AND STATUS LAMPS or CHECK FOR PROPER VALVE ACTUATION	
<u>Normal Pumping Messages</u>	<u>Action</u>
LEFT (RIGHT) SYSTEM READY FOR USE	Refer to Section 2.1
LEFT (RIGHT) RATE XXX BPM OUTFLOW XX.X L/M	Refer to Section 2.5
TO STOP PUMPING, PRESS "OFF" AGAIN	Refer to Section 2.5.3
<u>Weaning Messages</u>	<u>Action</u>
To set flow, press and hold ACTIVATE	Refer to Section 2.7.1
Turn SELECT to set flow: XX.X L/M	Refer to Section 2.7.1
LEFT (RIGHT) RATE XXX BPM WEANING @ XX.X L/M	Refer to Section 2.7.1
Not enough flow... TURN OFF WEANING	Turn off weaning select knob
<u>Alarm Messages</u>	<u>Action</u>
LOW FLOW LOW PRESSURE HIGH PRESSURE	Check hoses for leaks or kinks. (Section 3.2)
WARNING: FOOT PUMP HANDLE IS DISLODGED	Swing down transfer lever
SYSTEM FAILURE Obtain backup console immediately	Get backup console.

This page left intentionally blank

OPERATING INSTRUCTIONS

Normal	<ol style="list-style-type: none"> 1) Turn power "ON" (switch and line cord located at rear). 2) Wait for display: "LEFT (RIGHT) SYSTEM READY FOR USE" 3) Connect pneumatic line(s) from disposable at rear. 4) To start pumping, depress appropriate "ON" button. 5) To stop pumping, depress "OFF" button twice.
Weaning	<ol style="list-style-type: none"> 1) Access WEANING CONTROLS. Turn SELECT knob on. 2) Depress ACTIVATE button while turning SELECT knob until desired flow is displayed. 3) Release ACTIVATE button. 4) For normal operation, turn SELECT switch to OFF position.
Battery	Battery, activated by disconnecting line cord, provides approximately one hour of operation when charged. With AC power on, BATTERY CHARGING message is displayed if charge is less than 80% capacity.

ALARMS**Alarm Messages****Probable Causes**

LOW FLOW

Obstruction of blood lines
 Blood pump placed too high relative to atrium
 Inadequate blood volume

LOW PRESSURE
LOW FLOW

Disconnection or leak in air line

HIGH PRESSURE
LOW FLOW

Air line or blood pump lines kinked or occluded

EMERGENCY SYSTEM ON

Microprocessor system failure:
obtain backup BVS console.

BATTERY ON LOW

Low batteries (<10 min.of battery power): Plug machine in or obtain backup BVS console.

SYSTEM FAILURE
(continuous audible alarm)

Remove auxiliary pump located in rear compartment and continue pumping by foot while observing blood pump(s). Obtain backup BVS Console

BATTERY CHARGING LOW

Battery charging fault; obtain backup BVS console.

Appendix A

Considerations when swapping between a BVS® 5000i Console and an AB5000™ Console for patients supported on a BVS® Blood Pump

Before changing from a BVS 5000i Console (BVS 5000i) to an AB5000 Console (AB5000), the operation of the blood chambers should be checked. If the atrial and ventricular chambers are not filling completely with each beat, pump height and volume status should be adjusted. If the Console changeover is completed without ensuring that the chambers are filling completely, there may be a drop in the **displayed** flow rate by as much as 1.5 L/min from the BVS 5000i to the AB5000. Stability of the patient after the console changeover can be used to confirm that the **actual** flow rate delivered to the patient has not been affected by changing to the AB5000.

The BVS Blood Pump Instructions for Use (document 0505-9000) provides the following instructions for assessing blood pump motion:

“Observe the filling and emptying of the upper bladder after the pump as reached a steady state (approximately 5 minutes after the start of pumping). The upper bladder should fill completely and then immediately begin to empty. If this “fill-to-empty” motion is not observed, the pump may be lowered to improve filling.”

Obtaining proper chamber (bladder) filling is crucial to maintain the accuracy of the displayed flow rate because the Consoles calculate the blood flow based on the airflow returning from the pump during diastole. These calculations are most accurate when both chambers are filling completely with each beat. If the flow rate into the device is 4 L/min or less, the Console may over report flow rate by as much as 1.5 L/min when atrial and ventricular bladders are not filling completely.

The BVS 5000i tends to over predict flow under inflow limited conditions (inflow cannula obstruction, low volume, pump placed too high) to a much greater extent than the AB5000. This difference between Consoles can result in the observation of a decrease in flow rate when changing from a BVS 5000i to an AB5000.

For example, the BVS 5000i may display a flow rate of 5.0 L/min when the measured (actual) flow rate delivered to the patient is 3.5 L/min **if the lower chamber is not filling completely**. If a patient is changed over to an AB5000 under this condition, the AB5000 may display a flow rate of 3.6 L/min **because it is better at maintaining a full lower bladder under inflow limited conditions**. The difference in displayed flow is a result of the inaccuracy of the BVS 5000i under this operating condition, not a difference in the flow delivered by the Console.

ABIOMED, Inc.

BVS 5000 DOMESTIC LIMITED SERVICE WARRANTY

ABIOMED, Inc. warrants that, at the time of installation, all BVS 5000 consoles (the "Goods") sold will be free from defects in material and workmanship and remain free from defects under normal use and service for a period of one (1) year from date of installation. Extended warranty and service may, at ABIOMED's option, be offered for an additional charge, in which event separate or additional terms and conditions apply.

The express warranty set forth on this page is the only warranty given by ABIOMED with respect to any goods furnished hereunder, ABIOMED makes no other warranty, express, implied or arising by custom or trade usage, and specifically makes no warranty of merchantability or of fitness for any particular purpose. Said express warranty shall not be enlarged or otherwise affected by ABIOMED's rendering of technical or other advice or service in connection with the Goods.

ABIOMED shall not be liable for incidental or consequential losses, damages or expenses, directly or indirectly arising from the sale, handling or use of the Goods, or from any other cause relating thereto, and ABIOMED's sole responsibility under this warranty will be, at its option, to 1) repair or replace the Goods or any component of the Goods found to be defective in workmanship or material during the foregoing warranty period, or 2) to refund the purchase price paid. All replaced components and Goods will become the property of ABIOMED. This warranty shall not apply if the Goods have been: (a) repaired or altered in any way by other than ABIOMED or ABIOMED authorized service personnel; (b) subjected to physical or electrical abuse or misuse; or (c) operated in a manner inconsistent with ABIOMED's instructions for use of the Goods. If ABIOMED determines that a claim was not caused by ABIOMED or ABIOMED's authorized service personnel, then Buyer shall pay ABIOMED for all related costs incurred by ABIOMED. This warranty is not transferable without the express written consent of ABIOMED.

Under this warranty, ABIOMED will provide at no charge, updates or modifications which directly affect the safe operation of the Goods. ABIOMED is not obligated to provide updates or modifications which provide (a) product improvement or enhancement; (b) new product features, or (c) options to the Goods.

ABIOMED has no obligation to provide a loaner system during service or maintenance of the Goods, however, at ABIOMED's sole discretion, ABIOMED may provide such loaner systems.

This warranty applies to the BVS 5000 console only and not to any disposable or other component of the BVS 5000 system. Specific items excluded from this warranty include, but are not limited to, blood pumps, cannula, and external tubing.

This warranty may not be amended without the express written consent of an authorized officer of ABIOMED, Inc.

U.S. Patents 4,782,817 and 4,826,477